



The Relief Bag is made from 250 micron thickness PVC and a Silicon tube.
 BS EN ISO 8669-2:1997 Compliant.
 Capacity: 1.6L.

- Instructions for Use (See reverse of bag for detailed instructions):
1. Open the clip, squeeze the narrow end of the Whiz® together and push it 1 cm into the Relief Bag tube. It will be a tight fit.
 2. Position the Relief Bag below the Whiz® and wee normally through the Whiz®
 3. When finished firmly close the clip and remove the Whiz®
 4. Dispose of urine at your first opportunity after use.

A Continence Product
 CE marked (MDD 93/42/EC)
 Authorised Representative:
 Freedomlast Ltd, Park Manor,
 Upper Malow St. Limerick, Ireland
 www.whizaway.com
 Whiz is a registered trademark of Jbol Ltd.
 Manufactured & Distributed by:
 Jbol Limited, 1 Folly Bridge
 Oxford OX1 4LB, UK
 Tel: +44 (0) 1865 240572
 Fax: +44 (0) 1865 246565
 www.jbol.co.uk
 info@jbol.co.uk

The Relief Bag can be washed out and re-used. It is primarily designed for emergency and one-off situations and should not be re-used on a regular basis over an extended period of time.



www.whizfreedom.com



whiz® Relief bag

The Relief bag is a urine collection bag for use with the Whiz® freedom. It is ideal for emergencies, traffic jams, medical needs, or simply when it's too cold to go outside!

The Relief bag attaches to the Whiz® freedom and all known catheters. The Whiz® freedom is used by women of all ages, all over the world, for travel, sport, convenience and medical needs. It allows women to wee standing, sitting, crouching or lying-down without the need to undress. It resolves the problems of unclean or unavailable toilets.

For full range of the Whiz® products visit:
www.whizfreedom.com

