



Cleancatch® Midstream

whiz®

for when health calls
Urine Collection Device





For the Healthy, Accurate and Hygienic Collection of a Midstream Urine Sample
 Pour le recueil sain, précis et hygiénique de l'urine en milieu de jet
 Kollektor zur hygienischen Abnahme einer Mittelstrahl-Urinprobe
 Per il prelievo igienico, preciso e sano di un campione di urina midstream
 Para la recogida sana, precisa e higiénica de muestras de orina a mitad de la micción
 Para uma colheita saudável, precisa e higiénica de amostras da parte intermédia da micção

▲
 Peel to Open
 Soulever pour ouvrir
 Hier aufreißen
 Strappare per aprire
 Despegue para abrir
 Destaque para abrir

Single Use Only
 Réservé à un usage unique
 Nur zur einmaligen Verwendung
 Monouso
 Para un solo uso
 Não reutilizável

Procedure for use
Mode d'emploi
Gebrauchsanleitung

Istruzioni per l'uso
Procedimiento de uso
Procedimento de utilização

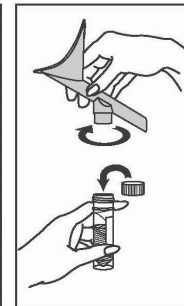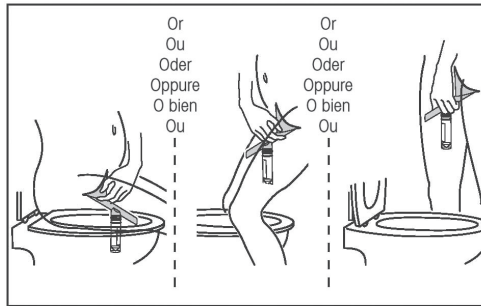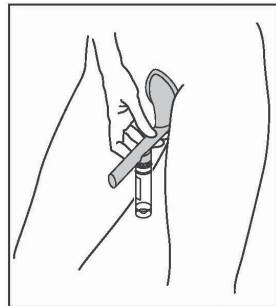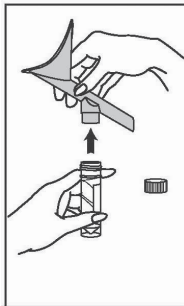

External Use Only
 Réservé à un usage externe
 Nur für den externen Gebrauch
 Per solo uso esterno
 Exclusivament e para uso externo
 Apenas para uso externo

ENG	Attach Bottle	Hold Cleancatch against body	Pass urine normally, do not stop and start	Twist off to detach bottle, twist on bottle cap
FRA	Fixer le flacon	Placer Cleancatch contre le corps	Uriner normalement, nul besoin de s'arrêter et de reprendre	Dévisser pour détacher le flacon, visser le bouchon du flacon
GER	Probenflasche anschrauben	Cleancatch an den Körper halten	Normal Urin lassen; es ist nicht nötig, den Urinstrom zu unterbrechen	Probenflasche abschrauben, Verschluss aufschrauben
ITA	Collegare il flacone	Reggere Cleancatch contro il corpo	Urinare normalmente. Non è necessario arrestare la minzione e riprenderla	Svitare il flacone per staccarlo, avvitare il tappo
SPA	Conecte el frasco	Sujete el dispositivo Cleancatch contra el cuerpo	Orine normalmente, sin necesidad de detenerse y continuar	Desenrosque para retirar el frasco, enrosque la tapa del frasco
POR	Aplique o frasco	Mantenha o Cleancatch contra o corpo	Urine normalmente, não necessita de parar e começar	Rode para destacar o frasco, coloque a tampa no frasco

Contents: One Cleancatch collection device, one sample bottle and cap
 Contenu: un appareil de recueil Cleancatch, un flacon de prélèvement et un bouchon
 Inhalt: 1 Cleancatch Urinkollektor, 1 Probenflasche mit Verschluss
 Contenuto: un dispositivo di raccolta Cleancatch, un flacone per campione ed un tappo
 Contenido: Un dispositivo colector Cleancatch, un frasco para muestras y su tapa
 Conteúdo: Um dispositivo de colheita Cleancatch, um frasco de colheita e tampa

Sterile if unopened
Stérile si l'emballage n'est pas ouvert
Im ungeöffneten Zustand steril
Sterile se sigillato
Estéril si no se ha abierto
Estéril, se estiver por abrir

Do not touch inside of device or sample bottle
 Ne pas toucher l'intérieur du dispositif ou du flacon de prélèvement
 Die Innenseiten des Kollektors und der Probenflasche nicht berühren!
 Non toccare l'interno del dispositivo o del flacone del campione
 No toque el interior del dispositivo o del frasco para muestras
 Não toque no interior do dispositivo ou do frasco de colheita

Dispose of device as instructed
 Jeter le dispositif selon les instructions
 Kollektor anweisungsgemäß entsorgen
 Smaltire il dispositivo nel modo indicato
 Deseche el dispositivo según las instrucciones
 Descarte o dispositivo de acordo com as instruções

Manufactured and Distributed by:
 Jbol Limited
 1 Folly Bridge, Oxford OX1 4LB, UK
 Tel: +44 (0) 1865 240572
 www.jbol.co.uk
 Patents and Design Rights granted and protected



Authorised Representative:
 Freedomatlast Ltd
 David O'Donnell and Associates
 Second Floor
 Park Manor
 Limerick, Ireland

Reference Code:
C730: Whiz Cleancatch Midstream (30ml bottle)
C7B30: Whiz Cleancatch Midstream with Boric Acid - red cap (30ml bottle)
C710: Whiz Cleancatch Midstream (10ml bottle)
C7B10: Whiz Cleancatch Midstream with Boric Acid - red cap (10ml bottle)

 See date stamp